

HP handle

Carl-Oskar och Rebel

Art. nr: 22115, HP-handle left, one hand brake right

22116, HP-handle right, one hand brake left

When replacing parts on the HP handle, only Human Care's original parts must be used for the CE marking to apply.



Assembly

Usage

The HP handle is designed for users who need extra support when walking.

Observe!

The HP handle may only be used for Carl-Oskar and Rebel rollators with a width of 62, 67 and 72 cm.

When assembling the HP handle, the associated one-hand brake must be assembled.

IMPORTANT!

The HP handle may only be adjusted so that the center of gravity is within the rollator's wheel base.

Adjustment - Handle

Adjust the position of the handle. Fix it by tightening the wing knob.

Adjustment - Side

Adjust the HP handle laterally by turning it. Fix it by tightening the wing knob.

Adjustment - Side/Elevation

Adjust the HP handle height (see MAX marking on the tube) and lateral angle. Fix it by tightening the wing knob

IMPORTANT! The angle of the handle tube must not point outwards

